

## Hygiene<sup>PLUS</sup> Value Pack (Art.no. 891192)

Elacin Hygiene<sup>PLUS</sup> Value Pack for a thorough monthly cleaning of your hearing protectors.

### This Hygiene kit holds the following products:

- + 1 filter exchange tool
- + 1 cleaning box
- + 1 air blower
- + 12 cleaning tablets
  
- + 1 Elacin Hygiene kit
  - 1 wax remover tool + brush
  - 1 towel (for daily and monthly cleaning)
  - 2 sprays
  - 4 brushes
  - 1 user manual




As well as straightforward daily cleaning, we also recommend you to clean your hearing protectors thoroughly every month.

It is desirable, but not essentially necessary, to remove the acoustic filters before cleaning<sup>\*)</sup>. When the hearing protector is cleaned with the filter the attenuation can be slightly higher the first 24 hours after cleaning. After 24 hours of drying the filter has again an optimal performance.

<sup>\*)</sup> For Elacin ER > remove the filters before cleaning <sup>\*)</sup> For Elacin CH30 > don't remove the filters before cleaning, as they can't be removed,

Dry the outside of the hearing protectors with the towel and use the air blower to dry the sound canal. Once you are sure that the sound canal is completely dry, put the filters back into the hearing protectors and store the hearing protectors in the case provided.

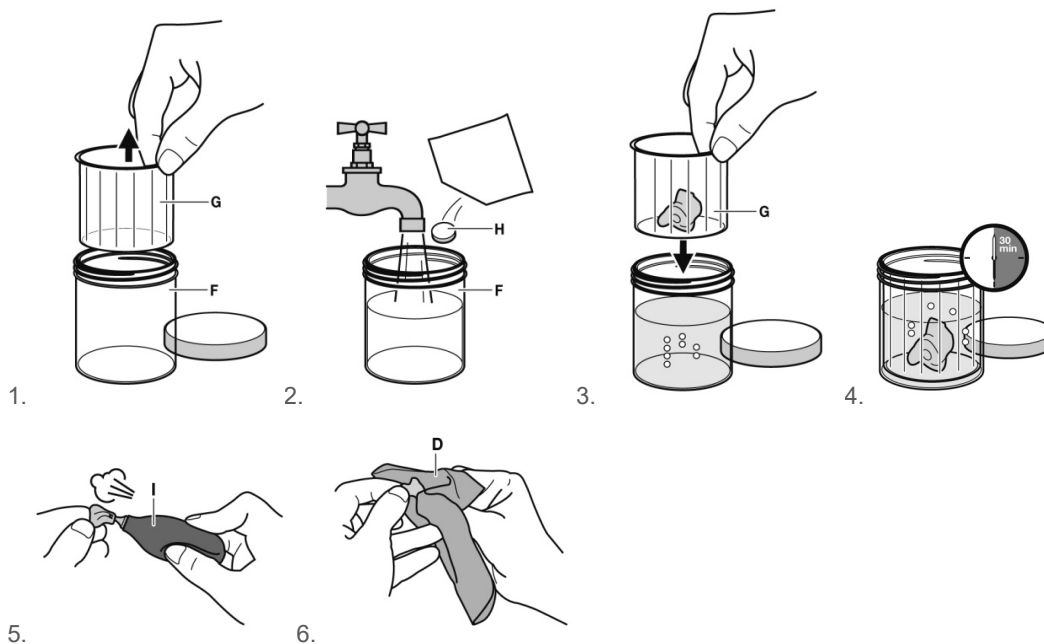

More information about the cleaning of your hearing protectors and the use of the cleaning products can be found in the user manual.