Elacin


RC-15 | RC-17 | RC-19 | RC-21
RC-23 | RC-25 | RC-27 | RC-29

EN | User Manual
NL | Gebruiksaanwijzing
DE | Gebrauchsanleitung
FR | Mode d'emploi
ES | Manual del usuario
IT | Manuale dell'utente
PT | Manual do utilizador
DK | Brugervejledning
FI | Käyttöohje
SE | Bruksanvisning
NO | Brukerveiledning
RO | Manual de utilizare
PL | Instrukcja obsługi
HU | Használati útmutató


Custom-made hearing protection
Op-maat-gemaakte gehoorbescherming
Maßgefertigter Gehörschutz
Protection auditive sur mesure
Protetores auditivos personalizados
Otoprotettori personalizzati
Proteção auricular feito à medida
Specialfremstillede ørepropper
Räätälöidyt kuulosuojaimet
Skräddarsydd hörselskydd
Skreddersydd hørselsvern
Protecție auditivă personalizată
Wkładki przeciwhałasowej wykonanej na wymiar
Egyedi készítésű hallásvédő

1.	EN		User Manual
2.	NL		Gebruiksaanwijzing
3.	DE		Gebrauchsanleitung
4.	FR		Mode d'emploi
5.	ES		Manual del usuario
6.	PT		Manual do utilizador
7.	IT		Manuale dell'utente
8.	DK		Brugervejledning
9.	FI		Käyttöohje
10.	SE		Bruksanvisning
11.	NO		Bruerveiledning
12.	RO		Manual de utilizare
13.	PL		Instrukcja obsługi
14.	HU		Használati útmutató


Elacin RC-15 | RC-17 | RC-19 | RC-21 | RC-23 | RC-25 | RC-27 | RC-29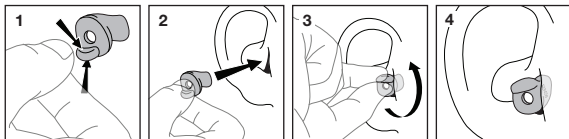

Elacin RC-15 | RC-17 | RC-19 | RC-21 | RC-23 | RC-25 | RC-27 | RC-29 minigrip

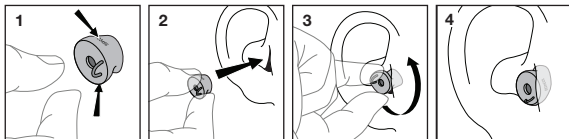Elacin RC-15 | RC-17 | RC-19 | RC-21 | RC-23 | RC-25 | RC-27 | RC-29 no grip

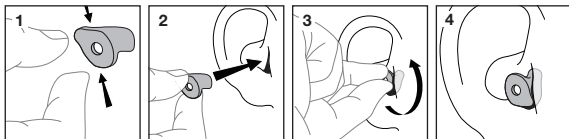Elacin RC Attenuation characteristics - Following EN352-2:2020

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f in Hz	Frequency in Hertz.
APVf in dB	Assumed Protection Value ($M_f - 1 \times S_f$), at given frequency f in decibel.
L in dB	Attenuation value in predominantly low-frequency noise spectrum ($f < 500$ Hz).
M in dB	Attenuation value in predominantly medium-frequency noise spectrum ($f > 500$ and $f < 2000$ Hz).
H in dB	Attenuation value in predominantly high-frequency noise spectrum ($f > 2000$ Hz).
SNR in dB	Single Number Rating, to be deducted from C-weighted noise level.
Mf in dB	Average attenuation value at given frequency f in decibel.
L_m in dB	Mean attenuation in predominantly low-frequency noise spectrum ($f < 500$ Hz)
M_m in dB	Mean attenuation in predominantly medium-frequency noise spectrum ($f > 500$ Hz and $f < 2000$ Hz)
H_m in dB	Mean attenuation in predominantly high-frequency noise spectrum ($f > 2000$ Hz)
SNR_m in dB	Mean Single Number Rating
Sf in dB	Standard deviation at given frequency f in decibel.
L_s in dB	Standard deviation on attenuation in predominantly low-frequency noise spectrum ($f < 500$ Hz)
M_s in dB	Standard deviation on attenuation in predominantly medium-frequency noise spectrum ($f > 500$ Hz and $f < 2000$ Hz)
H_s in dB	Standard deviation on attenuation in predominantly high-frequency noise spectrum ($f > 2000$ Hz)
SNR_s in dB	Standard deviation on Single Number Rating

Specifications: The European type certification according to the Regulation 2016/425 was carried out by PZT GmbH, NB # 1974, Bismarckstraße 264 B, 26389 Wilhelmshaven, Germany.

The production control according to the EU regulation 2016/425 module D was carried out by QUINTIN CERTIFICATIONS, NB # 2927, 825 Route de Romans, 38160 Saint Antoine l'Abbaye, France. Declarations of conformity at www.elacin.com/declarations-of-conformity/

f in Hz	Frequentie in Hertz.
APVF in dB	Gemiddelde dempingswaarde per frequentie f ($M_f - 1 \times S_f$) in decibel (Assumed Protection Value).
L in dB	Gemiddelde dempingswaarde in overwegend laagfrequent geluidsspectrum ($f < 500$ Hz).
M in dB	Gemiddelde dempingswaarde in overwegend middenfrequent geluidsspectrum ($f > 500$ en $f < 2000$ Hz).
H in dB	Gemiddelde dempingswaarde in overwegend hoogfrequent geluidsspectrum ($f > 2000$ Hz).
SNR in dB	Dempingswaarde, af te trekken van C-gewogen geluidsniveau.
Mf in dB	Gemiddelde dempingswaarde bij gegeven frequentie f in decibel.
L_m in dB	Gemiddelde demping in overwegend laagfrequent geluidsspectrum ($f < 500$ Hz)
M_m in dB	Gemiddelde demping in een overwegend middenfrequent geluidsspectrum ($f > 500$ Hz en $f < 2000$ Hz)
H_m in dB	Gemiddelde demping in overwegend hoogfrequent geluidsspectrum ($f > 2000$ Hz)
SNR_m in dB	Gemiddelde dempingswaarde
Sf in dB	Standaarddeviatie bij gegeven frequentie f in decibel.
L_s in dB	Standaardafwijking van de demping in een overwegend laagfrequent geluidsspectrum ($f < 500$ Hz)
M_s in dB	Standaardafwijking van de demping bij een overwegend middenfrequent geluidsspectrum ($f > 500$ Hz en $f < 2000$ Hz)
H_s in dB	Standaardafwijking bij demping in overwegend hoogfrequent geluidsspectrum ($f > 2000$ Hz)
SNR_s in dB	Standaardafwijking van de dempingswaarde

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Elacin Custom-made hearing protection

IMPORTANT: Read this instruction carefully before use
and keep both the package and the user manual.

Intended use

This hearing protector is reusable and intended to protect the wearer from hazardous noise levels when fitted in accordance with this user instruction. Any other use is not intended and therefore not allowed.

Elacin quality

Elacin custom-made hearing protectors are subject to the highest quality standard in its industry. Each product is unique and manufactured with maximum care to guarantee safety and comfort. Each pair of hearing protectors is issued with an identification number.

Before use

Check that the hearing protectors are clean and dry before you use them. The hearing protector with the letter 'L' is for the left ear. The hearing protector with the letter 'R' is for the right ear. Your hearing specialist will be glad to demonstrate the correct way to insert and remove the hearing protectors.

Insertion (see illustration)

Insertion may be less smooth at first use. Application of included ear cream on the part of the hearing protector that goes into the ear will improve ease of insertion.

1. Hold the hearing protector's handle/ring between your thumb and index finger, with your thumb against the filter.
2. Turn the hearing protector such that your thumb is on top and hold it by your ear.
3. Insert the hearing protector by rotating it backwards and, using gentle pressure, putting it into the ear canal. The handle/ring is now at the top and horizontal, and your thumb is pointing downwards.
4. Using gentle pressure from your index finger (on the handle), move the hearing protector up and down slightly until it seals comfortably.

Removal

Remove the hearing protector using a rotary forward motion.

Or, alternatively, place your index finger against the lower part of your ear and press upwards a number of times. The hearing protector is then slowly released from the ear. When it no longer seals you can safely remove it.

Storage

Store the hearing protector in the supplied pouch when not in use.

Do not store above 55 degrees Celsius.

Maintenance

Regularly inspect your Elacin hearing protectors for serviceability. For a thorough cleaning of your custom-made hearing protectors, we advise you to use the Elacin Hygiene products.

With these cleaning products you keep your hearing protectors in the best possible shape and help to prolong the life-span. Ask your hearing specialist for more information.

Warning

- Use Elacin hearing protectors carefully in accordance with the instructions. If the instructions above are not followed, the protection certainty of the hearing protectors cannot be guaranteed.
- Use your Elacin hearing protectors always in noisy situations!
- NOTE: the protection is reduced substantially even if you only remove them for a very short period. You will not be adequately protected if you fail to wear the hearing protectors at all times.
- Changes in body weight can affect the fit of a hearing protector to the extent that it will not fully close off the hearing canal.
- This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
- The hearing protectors are custom-made and are therefore not transferable.
- The hearing protectors are designed to be inserted in the hearing canal only.
- Dirty hearing protectors might cause an ear infection, please follow maintenance instructions.
- If applicable: These hearing protectors should not be used where there is a risk that the connecting

cord could be caught up during use, for example, by a machine.

- Do not remove your hearing protectors by pulling the interconnecting cord.
- Sudden or fast removal of the hearing protectors out of the ear canal may damage the ear drum.
- Contact your supplier if you notice anything out of the ordinary.

Lifetime

Replace your hearing protectors after 4 years of use.

The date ("Replace within one year from ...") can be found on the customer card.

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Elacin is a registered trademark of Elacin International B.V.
Elacin International B.V.